



## Instructions Klearedge Polisher (2500-P)

**WARNING:** For DRY POLISHING ONLY! Do not use any liquid, electric shock might occur.

1. Place Machine on bench and plug into a 120 VAC outlet.
2. Charge disk with polishing bar by lightly rubbing it on the moving felt disk. Do NOT over charge. It is normal for loose particles to fly out as you use the felt disk for the first time.
3. After the disk has been charged, hold the lens with both hands. Support your hands by resting them on the rim of the machine, then bring lens in contact with the rotating disk, slowing moving it from side to side.
4. If lens is pressed too hard, a brown mark will result on the rotating disk.
5. If polishing material builds up on the lens, the disk is over charged and will take longer to polish.
6. The spot being polished can be seen through the inside of the lens.
7. It takes about 2-minutes to polish the lens.
8. Recharge disk only when it takes too long to polish. Care must be taken not to over charge the disk, for it will slow down the process.
9. Using any other polishing materials may result in poor quality finish or damage to disk.

### **Replacement Accessories:**

2502 - Replacement Felt Polish Pad

2503 - Replacement All Materials Polish Bar



**PRACTICAL SYSTEMS, INC.**  
11617 Prospect Road • Odessa, FL 33556  
(800) 237-8154 • Fax (800) 330-3800  
[www.LookToPSI.com](http://www.LookToPSI.com)